

# Product Bulletin

## For Distribution To:

- International Distributors
- International Management
- U.S. Field Sales & Clinical Specialists
- U.S. Technical & Field Service

Date: **January 4, 2013**

## RE: **Intercept® Detergent Premixing for Daily Use**

Some customers have inquired if Intercept can be mixed with water at the start of the work day, so that it is ready for immediate use throughout the course of the day. MEDIVATORS Intercept detergent can be premixed in a clean container for daily use. Intercept is supplied as a concentrate and when prepared to the proper use dilution, maintains its cleaning effectiveness and will not become contaminated with microorganisms for the duration of the work day.

When preparing Intercept for daily use in a clean one-gallon sized bottle, use Intercept at 1/3 ounce per gallon of water (0.25% concentration). If visible organic matter is present on the device(s) being cleaned, then use Intercept at 2/3 ounce per gallon of water (0.5% concentration). Rinse all surfaces and internal channels thoroughly with water after use.

A copy of this product bulletin is available on the MEDIVATORS website located at [www.medivators.com](http://www.medivators.com) under "Documents" section for Intercept Detergent. If you have any questions regarding this Product Bulletin, please contact MEDIVATORS Customer Support at 1-800-444-4729.



MEDIVATORS Inc. Headquarters  
14605 28th Avenue North  
Minneapolis, MN  
55447-4822 USA  
Tel: +1.800.444.4729  
Fax: +763.553.3387

MEDIVATORS BV  
Sourthweg 11  
6422PC Heerlen  
The Netherlands  
Tel: +31.45.5.471.471  
Fax: +31.45.5.429.695

MEDIVATORS Asia/Pacific Pte Ltd  
1 A International Business Park  
#05-01 Singapore 609933  
Tel: +65.6227.9698  
Fax: +65.6225.6848

MEDIVATORS Beijing Representative Office  
Room 708, 7th Floor Kaiheng Center, Block B  
No. 2 Chaoyangmennei Street  
Dongcheng District, Beijing China 100010  
Tel: +8610.6567.8446  
Fax: +8610.6567.8445

[www.medivators.com](http://www.medivators.com)