

Product Bulletin

For Distribution To:

- International Distributors
- International Management
- U.S. Field Sales & Clinical Specialists
- U.S. Technical & Field Service

Date: **January 4, 2013**

RE: **Intercept® Detergent Compatibility with Simple2 detergent**

We have received requests for a compatibility statement from MEDIVATORS indicating that Intercept is safe to use with Simple2 enzymatic detergent.

MEDIVATORS Intercept detergent was laboratory tested and determined to be chemically compatible with Simple2™ enzymatic detergent. Simple2 detergent is used in the “First Step” product offered by Cygnus Medical.

Laboratory testing was conducted by MEDIVATORS R&D Department between Intercept and the enzymatic detergent. The detergents were mixed together at recommended use concentration levels, as stated in the manufacturers’ directions for use. The mixed detergent solution was monitored and tested for a chemical reaction, temperature change, color change, and precipitation. A reaction or change in any of these categories would identify a non-compatibility condition between the tested detergents.

The laboratory testing resulted in no occurrence or change in the chemical, temperature, color and precipitation areas, and confirms that Intercept is compatible with the enzymatic detergent.

A copy of this product bulletin is available on the MEDIVATORS website located at www.medivators.com under “Documents” section for Intercept Detergent. If you have any questions regarding this Product Bulletin, please contact MEDIVATORS Customer Support at 1-800-444-4729.



MEDIVATORS Inc. Headquarters
14605 28th Avenue North
Minneapolis, MN
55447-4822 USA
Tel: +1.800.444.4729
Fax: +763.553.3387

MEDIVATORS BV
Sourthweg 11
6422PC Heerlen
The Netherlands
Tel: +31.45.5.471.471
Fax: +31.45.5.429.695

MEDIVATORS Asia/Pacific Pte Ltd
1 A International Business Park
#05-01 Singapore 609933
Tel: +65.6227.9698
Fax: +65.6225.6848

MEDIVATORS Beijing Representative Office
Room 708, 7th Floor Kaiheng Center, Block B
No. 2 Chaoyangmennei Street
Dongcheng District, Beijing China 100010
Tel: +8610.6567.8446
Fax: +8610.6567.8445

www.medivators.com