



FILTER PACKAGES

Advantage Plus Complete Filter Package - MF01-0072

Filter package includes: Air filter (1/pkg of 8), Basin drain filter (2/pkg of 12), 0.4µ Pre-filter (4), 1µ Pre-filter (4) and 0.1µ Bacterial-retentive absolute filter (2).

FILTERS

Air Filter (8 filters/box) - MF01-0028

Change filters on both sides every 6 months. Filter pkg will last approximately 12 months.

Basin Drain Filters (12 filters/box) - MF01-0059

Change once per month. They are located inside of each basin at the drain. Remove these filters with: **Basin Drain Filter Hook Part # 5-6-170.**

0.1 Micron Absolute Bacteria Retentive Water Filter (one each) - MF01-0069

This filter is located outside the Advantage Plus. When looking at the pre-filtration it is located on the far left. It needs to be changed every 6 months or sooner if poor water conditions exist. Always keep one on hand and a water line disinfectant process must be performed when changing this filter.

0.4 Micron Water Filter (one each) - MF01-0071

This filter is located outside the Advantage Plus. When looking at the pre-filtration system, it is the filter in the middle of the pre-filtration system in between the 1 micron and the 0.1 micron. Change is pressure related, water pressure must be 40 psi or above. Always keep one on hand. Change at least every 3 months even if pressure remains above 35 psi.

1 Micron Water Filter (one each) - MF01-0070

This filter is located outside the Advantage Plus. When looking at the pre-filtration system, it is the filter located on the far right. Change is pressure related, water pressure must be 40 psi or above. Always keep one on hand. Change at least every 3 months even if pressure remains above 35 psi.

Active Vapor Management (Charcoal) Replacement Filter (one each) - MF01-0068

If an Active Vapor Management system (AVM) is installed, it is located behind the machine. The AVM pulls harmful chemical vapors away from the user and passes it through this carbon filter, which neutralizes the vapors. This filter needs to be changed every 6 months.

HIGH-LEVEL DISINFECTANT

Rapicide PA (Two - 5 liter bottles each of Part A and Part B/case) - ML02-0117

The Advantage Plus fits one each of Part A and Part B inside the system and each bottle connects to a specific uptake tube for accurate chemical mixing. Approximately 44 cycles per case. Always make sure to have an extra case on-hand. NOTE: Rapicide PA is considered hazardous material and cannot be shipped overnight (i.e. shipped by air). Please allow at least 5 business days for delivery.

Rapicide PA Indicator Test Strip (2 bottles/case, each bottle has 100 strips) - ML02-0118

Usage depends on how many procedures/day.

DETERGENT AND WIPES

Intercept Detergent (4 - one gallon bottles/case) - ML02-0106

Endoscope Detergent for pre-cleaning, proven to reduce biofilm. Use only one ounce (1 full pump) of Intercept Detergent in three (3) gallons of water.

Intercept Wipes (4 containers/case, each container has 100 wipes) - ML02-0107

Intercept detergent towelettes a low cost way to wipe down contaminated endoscopes and reduce biofilm.

MISCELLANEOUS

Printer Paper (one each) - 1-4-501 (6 pack) - 40100-104

Single ply with spindle hole. Usage is dependent on the volume of procedures.

Printer Ribbon (one each) - 1-4-503

Black/red ink printer ribbon. Usage is dependent on the volume of procedures.

Accessory Bags (set of 2) - CB12-0002

Accessory bag for placement of endoscope accessories, such as valves.

Barcode Tags (one set of #100-199) - 67199-157

Adhesive barcode tags include duplicates of each number.

Barcode Tags (one set of #200-299) - 67199-158

Adhesive barcode tags include duplicates of each number.

Barcode Tags (one set of #300-399) - 67199-541

Adhesive barcode tags include duplicates of each number.

Advantage Plus Sampling Cups (pkg / 100 cups) - ML02-0121

Used for testing the Rapicide PA disinfectant concentration. Usage is dependent on volume of procedures.

Silicone Oil (one each) - 17599-496

Used for lubricating connector O-rings. Lubricate O-rings at least once a week.

To order, call MEDIVATORS directly at: 1-800-444-4729, option 1 • www.minntech.com/medivators