# MEDIVATORS® INTERCEPT®

## Next-Generation Detergent WIPES

**REF ML02-0107**

Next generation detergent wipe for precleaning endoscope and other medical devices

### PACKAGING:

4 Canisters per Case  
100 Ore-saturated Towelettes per Canister

**READ COMPLETE DIRECTIONS FOR USE AND MSDS FOR INTERCEPT WIPES PRIOR TO USE. READ INSTRUMENT MANUFACTUER'S CLEANING INSTUCTIONS PRIOR TO USE.**

### EN - DIRECTIONS FOR USE

INTERCEPT Wipes cleaning towelette is a disposable towelette pre-saturated with INTERCEPT detergent. INTERCEPT Wipes are intended for use on endoscopes and other medical devices. For cleaning of appropriate medical devices, use one or two wipes to completely preclean the surface of the instrument of all gross debris. To start canister feed, carefully remove cover. Using forceps, pull up a towelette corner from center of roll, twist onto a point and thread through the hole in the conatiner cover. Pull through with forceps about 1 inch. Replace cover. Pull out first towelette and tear off at a 90 angle. Remaining towelettes feed automatically, ready for next use. When not in use, keep center cap closed to prevent solution loss.

**INTERCEPT WIPES IS NOT A SURFACE DISINFECTANT AND IS NOT INTENDED TO DISINFECT MEDICAL INSTRUMENTS. OR ACCESSORIES INSTRUMENTS THAT REQUIRE DISINFECTION, SUCH AS ENDOSCOPES AND ACCESSORIES, MUST STILL BE DISINFECTED IN ACCORDANCE WITH THE MANUFACTURER'S INSTRUCTIONS AFTER CLEANING WITH INTERCEPT.**

### DESCRIPTION:

- Patented formula will rapidly penetrate and solubilize blood, proteins, cellular mucosal debris, carbohydrates, lipids and mucopolysaccharides.

- Laboratory proven to effectively remove biological and organic residues from a variety of medical instruments and surfaces.

### DISPOSAL OF INFECTIOUS MATERIAL:

Cleaning materials used that may contain blood or other body fluids should be autoclaved and disposed of in accordance with Federal, State and Local regulations for infectious materials disposal.

### PRECAUTIONARY STATEMENTS:

Use of nitirle gloves or equivalent is recommended for all cleaning involving reusable medical devices and accessories. Refer to the Material Safety Data Sheet (MSDS) for additional information.

### DO NOT USE AS BABY WIPES.

### FIRST AID:

In case of contact with eyes, flush with copious amounts of water for fifteen (15) minutes. In case of ingestion, DO NOT induce vomiting. Contact CHEMTREC for emergency medical treatment at 1-800-424-9300.

### STORAGE & DISPOSAL:

Store at room temperature 15°C-25°C (59°F - 77°F). Discard unused towelettes according to local, state and federal regulations. DO NOT reuse container.

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