

ESC Endoscope Storage Cabinet

Maintenance & Cleaning Protocol

Daily Maintenance

- Check that the door alarms and locks are functioning properly and that the door over-ride key can be located.
- Verify air flow is exiting each hookup port.
- Check the internal cabinet pressure reading to ensure the system is maintaining pressure above 10 Pa when the cabinet is in use and the doors are closed.
- Monitor the absorbent matting for excessive moisture. Replace as needed

Weekly Maintenance

- Maintain the o-rings on the hookup connections by lubricating with silicone oil.
- Change the absorbent matting as required – minimum of weekly.

Monthly Maintenance

- Wipe down the inside of the cabinet with a disinfectant wipe.
- Ensure the air vents are not obstructed and dust free.
- Check the hookups for cracks, splits or tears in the tubing and replace as necessary. Ensure that all tubing is secure and connectors are also secure.
- If the hookups are not routinely used during High-Level Disinfection, wipe the surface of the hookups with a disinfectant wipe.