

MEDIVATORS™ TUBE CLIP

REF #100180

MEDIVATORS™ Tube Clip

Intended Use:

Tube clips are to be used in conjunction with the ENDO SMARTCAP®, ENDOGATOR®, or ENDOGATOR HYBRID Disposable Endoscopic Tubing. The tube clip is intended to be used when tubing is not in use.

Instructions for Use:

REF 100180 MEDIVATORS Tube Clip

1. Clean desired area of cart, or specific area, that tube clip is to be affixed.
2. Separate strip along perforated edge.
3. Remove **RED** liner from 3M Command Strip and press adhesive to clip.
4. Remove **BLACK** liner from 3M Command Strip.
5. Press and hold ten seconds to desired location of cart surface.
6. Once affixed, tubing may be placed in clip between procedures.

NON STERILE



www.medivators.com/ifus



Manufactured in the USA by:

Medivators Inc.
3150 Pollok Drive
Conroe, TX 77303
Tel: +1.936.539.0391

Medivators Inc.
14605 28th Avenue North
Minneapolis, MN 55447 USA
Toll Free: +1.800.444.4729

Medivators BV
Sourethweg 11
6422PC Heerlen
The Netherlands
Tel: +31.45.5.471.471

www.medivators.com

C.R. KENNEDY & COMPANY PTY. LTD.
300 LORIMER STREET
PORT MELBOURNE 3207
VICTORIA
AUSTRALIA
TEL: +61.3.9823.1555
FAX: +61.3.9823.1591

AUS

Australian
Sponsor Address

MEDIVATORS™, ENDO SMARTCAP®, and ENDOGATOR® are trademarks and registered trademarks of Medivators Inc.