

CAT NO. ML02-0107



READ MSDS FOR ADDITIONAL
PRECAUTIONARY STATEMENTS.

DISPOSAL OF INFECTIOUS MATERIAL:

Cleaning materials used that may contain blood or other body fluids should be autoclaved and disposed of in accordance with Federal, State and Local regulations for infectious materials disposal.

PRECAUTIONARY STATEMENTS:

Use of personal protective equipment including appropriate clothing, gloves, and safety glasses is recommended for all cleaning involving re-usable medical devices and accessories. Please refer to the Material Safety Data Sheet (MSDS) for additional information (www.minntech/medivators.com).

FIRST AID:

In case of contact with eyes, flush with copious quantities of water for 15 minutes. In case of ingestion, DO NOT induce vomiting. Contact Chemtrec for emergency medical treatment at 1-800-424-9300.

STORAGE & DISPOSAL:

Store at room temperature 15°C-25°C (59°F-77°F). Discard unused towelettes according to local, state, and federal regulations. Do not reuse container.

Intercept™ Wipes

Next generation detergent wipe for precleaning
endoscopes and other medical devices.

NET CONTENTS
100 Pre-saturated Towelettes
(7"x10")

KEEP OUT OF REACH OF CHILDREN
CAUTION

FEATURES

- Laboratory proven to effectively remove biological and organic residues from a wide variety of medical instruments and surfaces.
- Effective cleaning of medical instruments is the critical first step in a validated reprocessing program, and INTERCEPT can help to enhance that process.

CLEANING INSTRUCTIONS:

For cleaning of endoscopes and related accessories or other instruments, use one or two wipes to completely preclean the surface of the instrument of all gross debris.

DIRECTIONS FOR USE

Intercept Wipes cleaning towelette is a disposable towelette pre-saturated with Intercept. Intercept Wipes is intended for use on endoscopes and other medical devices.

DO NOT USE AS BABY WIPE!

DISPENSER DIRECTIONS:

To start feed, carefully remove cover. Using forceps, pull up a towelette corner from center of roll, twist it into a point and thread it through the hole in the container cover. Pull through with forceps about 1 inch. Replace cover. Pull out first towelette and tear off at a 90° angle. Remaining towelettes feed automatically, read for next use. When not in use, keep center cap closed to prevent solution loss.

Manufactured in the U.S.A. and Distributed by:



MEDIVATORS®
REPROCESSING SYSTEMS

A MINNTECH CORPORATION BUSINESS GROUP

Minntech Corporation
14605 28th Avenue North
Minneapolis, MN 55447 USA
Phone: (763) 553-3300
Toll Free: (800) 328-3340
Fax: (763) 553-2493
Website: www.minntech.com

U.S. Pat. No. 6,855,678 and foreign counterparts (solution)
© 2009, MINNTECH CORPORATION 67199-012/E