

**PRECAUTIONARY STATEMENTS:**

Use of personal protective equipment including appropriate clothing, gloves, and safety glasses must be worn. Refer to the Material Safety Data Sheet (MSDS) for additional information.

**DIRECTIONS FOR USE:**

For manual cleaning, use MEDICLEAN EZ at 1/2 oz. (15 mL) per gallon of water with one full stroke of the hand-pump (1 oz) to two gallons of water (0.39% concentration). For best cleaning results, mix MEDICLEAN EZ with water (do not exceed 45°C or 113°F) and ensure a contact time of (2) two minutes. Additional soak time may be needed for heavier bioburden loads or dried on soils. Rinse all surfaces and internal channels thoroughly with warm tap or distilled water. For use in automated washers or ultrasonic cleaners, follow equipment manufacturer's recommendations. Do not mix MEDICLEAN EZ with bleach or any other chemicals.

**FIRST AID:**

In case of contact with eyes, flush with copious amounts of water for 15 minutes. In case of ingestion, DO NOT induce vomiting. Contact CHEMTREC for emergency medical treatment at 1-800-424-9300.

**STORAGE & DISPOSAL:**

Store at room temperature 15°C-30°C (59°F-86°F) out of direct sunlight. MEDICLEAN EZ has a two (2) year shelf life. Discard unused solution according to local, state and federal regulations. DO NOT reuse container.

**LOT:****EXP:**

MEDIVATORS®

# MediClean EZ®

## Enzymatic Detergent

Proteolytic  
Cleaning Action

Formulated for cleaning surgical instruments,  
medical devices and endoscopes.

Use 1 pump with 2 gallons of water

**NET CONTENTS****1**

US Gallon  
(3.8 Liters)

**KEEP OUT OF REACH OF CHILDREN  
CAUTION**

**REF ML02-0131**

- Enzymatic presoak and cleaner formulated for use with medical instruments including immersible endoscopes, medical accessories and devices, surgical instruments and dental equipment.
- Proteolytic action for fast enzymatic cleaning of proteinaceous soils including blood, proteins, and other hard to remove soils including mucus, body tissue, carbohydrates, fats, lipids, and mucopolysaccharides.
- Low foaming, neutral pH, detergent which is free-rinsing and 100% biodegradable.
- Highly concentrated and cost effective; use only 1/2 ounce (15 mLs) per 1 gallon of water.
- For use in manual cleaning, automated washers, and ultrasonic cleaners.
- Intended for use in health care settings such as hospitals, surgical centers, neonatal units, medical clinics, emergency medical settings, laboratories, dental offices and veterinary facilities.



READ MSDS FOR ADDITIONAL PRECAUTIONARY STATEMENTS.

**MEDIVATORS**  
A Cantel Medical Company

Manufactured in the USA by:

MEDIVATORS Inc.  
14605 28th Avenue North  
Minneapolis, MN 55447 USA  
Toll Free: +1.800.444.4729

[www.medivators.com](http://www.medivators.com)

© 2013 MEDIVATORS Inc. 67199-632/B