MEDIVATORS®
**ENDO
 SMARTCAP®**
 Disposable Endoscopy Tubing

Reorder No. 100145 / 100145U/ 100145CO2/ 100145CO2U
ENDO SMARTCAP™ Tubing for OLYMPUS®140/240, 160/260, 180/280, 190/290 Series GI Endoscopes
 Bestellnr. 100145 / 100145U / 100145CO2 / 100145CO2U
 ENDO SMARTCAP™ Tubing for OLYMPUS®140/240, 160/260, 180/280, 190/290 Series GI Endoscopes
 Référence de réapprovisionnement n° 100145/100145U/100145CO2/100145CO2U
 ENDO SMARTCAP™ Tubing for OLYMPUS®140/240, 160/260, 180/280, 190/290 Series GI Endoscopes
 N.º de pedido 100145 / 100145U/ 100145CO2/ 100145CO2U
 ENDO SMARTCAP™ Tubing for OLYMPUS®140/240, 160/260, 180/280, 190/290 Series GI Endoscopes
 Cod. riordino 100145 / 100145U/ 100145CO2/ 100145CO2U
 ENDO SMARTCAP™ Tubing for OLYMPUS®140/240, 160/260, 180/280, 190/290 Series GI Endoscopes
 Nabestelnr. 100145 / 100145U/ 100145CO2/ 100145CO2U
 ENDO SMARTCAP™-slang voor GI endoscopen van de serie OLYMPUS®140/240, 160/260, 180/280, 190/290
 Bestillingsnummer 100145 / 100145U/ 100145CO2/ 100145CO2U
 ENDO SMARTCAP™-slange til OLYMPUS® serie 140/240, 160/260, 180/280, 190/290 GI-endoskoper
 Ombestillingsnr. 100145 / 100145U/ 100145CO2/ 100145CO2U
 ENDO SMARTCAP™-slanger for GI-endoskoper i serien OLYMPUS®140/240, 160/260, 180/280, 190/290
 Nova encomenda N.º 100145 / 100145U/ 100145CO2/ 100145CO2U
 Tubo ENDO SMARTCAP™ para endoscópios gastrointestinais OLYMPUS® das Séries 140/240, 160/260, 180/280, 190/290
 Beställningsnr. 100145 / 100145U/ 100145CO2/ 100145CO2U
 ENDO SMARTCAP™ slang för magtarmendoskop i OLYMPUS®-serierna 140/240, 160/260, 180/280, 190/290
 № для заказа 100145 / 100145U/ 100145CO2/ 100145CO2U
 Трубки ENDO SMARTCAP™ для ЖКТ эндоскопов серии OLYMPUS®140/240, 160/260, 180/280, 190/290
 Comandă reluată nr. 100145 / 100145U/ 100145CO2/ 100145CO2U
 Tubulatură ENDO SMARTCAP™ pentru endoscoape GI OLYMPUS® seriile 140/240, 160/260, 180/280, 190/290

Bestillingsnummer 100150 / 100150U/ 100150CO2/ 100150CO2U
 ENDO SMARTCAP™-slange til OLYMPUS® serie 100/200, 130/230 GI-endoskoper
 Ombestillingsnr. 100150 / 100150U/ 100150CO2/ 100150CO2U
 ENDO SMARTCAP™-slanger for GI-endoskoper i serien OLYMPUS® 100/200, 130/230
 Nova encomenda N.º 100150 / 100150U/ 100150CO2/ 100150CO2U
 Tubo ENDO SMARTCAP™ para endoscópios gastrointestinais OLYMPUS® das Séries 100/200, 130/230
 Beställningsnr. 100150 / 100150U/ 100150CO2/ 100150CO2U
 ENDO SMARTCAP™ slang för magtarmendoskop i OLYMPUS®-serierna 100/200, 130/230
 № для заказа 100150 / 100150U/ 100150CO2/ 100150CO2U
 Трубки ENDO SMARTCAP™ для ЖКТ эндоскопов серии OLYMPUS® 100/200, 130/230
 Comandă reluată nr. 100150 / 100150U/ 100150CO2/ 100150CO2U
 Tubulatură ENDO SMARTCAP™ pentru endoscoape GI OLYMPUS® seriile 100/200, 130/230

ENDO SMARTCAP™ Tubing for FUJIFILM™ GI Endoscopes
 Cod. riordino 100165 / 100165U/ 100165CO2/ 100165CO2U
 ENDO SMARTCAP™ Tubing for FUJIFILM™ GI Endoscopes
 Nabestelnr. 100165 / 100165U/ 100165CO2/ 100165CO2U
 ENDO SMARTCAP™-slang voor FUJIFILM™ GI endoscopen
 Bestillingsnummer 100165 / 100165U/ 100165CO2/ 100165CO2U
 ENDO SMARTCAP™-slange til FUJIFILM™ GI-endoskoper
 Ombestillingsnr. 100165 / 100165U/ 100165CO2/ 100165CO2U
 ENDO SMARTCAP™-slanger for FUJIFILM™ GI-endoskoper
 Nova encomenda N.º 100165 / 100165U/ 100165CO2/ 100165CO2U
 Tubo ENDO SMARTCAP™ para endoscópios gastrointestinais FUJIFILM™
 Beställningsnr. 100165 / 100165U/ 100165CO2/ 100165CO2U
 ENDO SMARTCAP™ slang för FUJIFILM™ magtarmendoskop
 № для заказа 100165 / 100165U/ 100165CO2/ 100165CO2U
 Трубки ENDO SMARTCAP™ для ЖКТ эндоскопов FUJIFILM™
 Comandă reluată nr. 100165 / 100165U/ 100165CO2/ 100165CO2U
 Tubulatură ENDO SMARTCAP™ pentru endoscoape GI FUJIFILM™

Reorder No. 100150 / 100150U/ 100150CO2/ 100150CO2U
ENDO SMARTCAP™ Tubing for OLYMPUS® 100/200, 130/230 Series GI Endoscopes
 Bestellnr. 100150 / 100150U / 100150CO2 / 100150CO2U
 ENDO SMARTCAP™ Tubing for OLYMPUS® 100/200, 130/230 Series GI Endoscopes
 Référence de réapprovisionnement n° 100150/100150U/100150CO2/100150CO2U
 ENDO SMARTCAP™ Tubing for OLYMPUS® 100/200, 130/230 Series GI Endoscopes
 N.º de pedido 100150 / 100145U/ 100145CO2/ 100145CO2U
 ENDO SMARTCAP™ Tubing for OLYMPUS® 100/200, 130/230 Series GI Endoscopes
 Cod. riordino 100150 / 100145U/ 100145CO2/ 100145CO2U
 ENDO SMARTCAP™ Tubing for OLYMPUS® 100/200, 130/230 Series GI Endoscopes
 Nabestelnr. 100150 / 100150U/ 100150CO2/ 100150CO2U
 ENDO SMARTCAP™-slang voor GI endoscopen van de serie OLYMPUS® 100/200, 130/230

Reorder No. 100160 / 100160U/ 100150CO2/ 100150CO2U
ENDO SMARTCAP™ Tubing for PENTAX® GI Endoscopes
 Bestellnr. 100160 / 100160U / 100150CO2 / 100150CO2U
 ENDO SMARTCAP™ Tubing for PENTAX® GI Endoscopes
 Référence de réapprovisionnement n° 100160/100160U/100150CO2/100150CO2U
 ENDO SMARTCAP™ Tubing for PENTAX® GI Endoscopes
 N.º de pedido 100160 / 100160U/ 100150CO2/ 100150CO2U
 ENDO SMARTCAP™ Tubing for PENTAX® GI Endoscopes
 Cod. riordino 100160 / 100160U/ 100150CO2/ 100150CO2U
 ENDO SMARTCAP™ Tubing for PENTAX® GI Endoscopes
 Nabestelnr. 100160 / 100160U/ 100150CO2/ 100150CO2U
 ENDO SMARTCAP™-slang voor PENTAX® GI endoscopen
 Bestillingsnummer 100160 / 100160U/ 100150CO2/ 100150CO2U
 ENDO SMARTCAP™-slange til PENTAX® GI-endoskoper
 Ombestillingsnr. 100160 / 100160U/ 100150CO2/ 100150CO2U
 ENDO SMARTCAP™-slanger for PENTAX® GI-endoskoper
 Nova encomenda N.º 100160 / 100160U/ 100150CO2/ 100150CO2U
 Tubo ENDO SMARTCAP™ para endoscópios gastrointestinais PENTAX®
 Beställningsnr. 100160 / 100160U/ 100150CO2/ 100150CO2U
 ENDO SMARTCAP™ slang för PENTAX® magtarmendoskop
 № для заказа 100160 / 100160U/ 100150CO2/ 100150CO2U
 Трубки ENDO SMARTCAP™ для ЖКТ эндоскопов PENTAX®
 Comandă reluată nr. 100160 / 100160U/ 100150CO2/ 100150CO2U
 Tubulatură ENDO SMARTCAP™ pentru endoscoape GI PENTAX®

Reorder No. 100141
ENDO SMARTCAP™ Adapter for FUJIFILM™ 530 Series Colonoscopes
 Bestellnr. 100141
 ENDO SMARTCAP™ Adapter for FUJIFILM™ 530 Series Colonoscopes
 Référence de réapprovisionnement n° 100141
 ENDO SMARTCAP™ Adapter for FUJIFILM™ 530 Series Colonoscopes
 N.º de pedido 100141
 ENDO SMARTCAP™ Adapter for FUJIFILM™ 530 Series Colonoscopes
 Cod. riordino 100141
 ENDO SMARTCAP™ Adapter for FUJIFILM™ 530 Series Colonoscopes
 Nabestelnr. 100141
 ENDO SMARTCAP™ Adapter for FUJIFILM™ 530 Series Colonoscopes
 Bestillingsnummer 100141
 ENDO SMARTCAP™-adapter til FUJIFILM™ serie 530 koloskoper
 Ombestillingsnr. 100141
 ENDO SMARTCAP™-adapter for koloskoper i FUJIFILM™ 530-serien
 Nova encomenda N.º 100141
 Adaptador ENDO SMARTCAP™ para colonoscópios FUJIFILM™ Série 530
 Beställningsnr. 100141
 ENDO SMARTCAP™ Adapter för kolonoskop i FUJIFILM™-serien 530
 № для заказа 100141
 Переходник ENDO SMARTCAP™ для колоноскопов серии FUJIFILM™ 530
 Comandă reluată nr. 100141
 Adaptor ENDO SMARTCAP™ pentru colonoscoape FUJIFILM™ eria 530

<p>INSTRUCTIONS FOR USE GEBRAUCHSANWEISUNG MODE D'EMPLOI INSTRUCCIONES DE USO</p>	<p>ISTRUZIONI PER L'USO GEBRUIKSIINSTRUCTIES BRUGSANVISNING BRUKSANVISNING</p>	<p>INSTRUÇÕES DE UTILIZAÇÃO BRUKSANVISNING ИНСТРУКЦИЯ ПО ПРИМЕНЕНИЮ INSTRUCȚIUNI DE UTILIZARE</p>
--	---	--

Intended Use:

ENDO SMARTCAP™ Tubing is intended to be used with an air or CO₂ source and / or pump along with a sterile water source to supply air or CO₂ and sterile water to a gastrointestinal endoscope during endoscopic procedures.

Warnings and Precautions:

- Guidelines for Preventing Cross Contamination:
 - ALWAYS prime the air / water channel prior to inserting the GI endoscope into the patient.
 - NEVER attach or reattach ENDO SMARTCAP™ Tubing to an GI endoscope that is / was inside the patient and has not been reprocessed.
 - ALWAYS close the pinch clip before disconnecting ENDO SMARTCAP™ Tubing from the GI endoscope.
 - NEVER turn off the air processor / CO₂ while ENDO SMARTCAP™ Tubing is attached to an GI endoscope.
 - ALWAYS remove the colonoscope from the light source before removing the adapter from the colonoscope. (#100141)
 - ALWAYS Dispose of the adapter after each procedure. (#100141)
 - NEVER Remove the backflow valve. Backflow valves are permanently attached to the device. (#100141)
 - NEVER Use a product with a missing backflow. If the backflow valve is missing, then discard the entire tubing, adapter, and water bottle. (#100141)
 - DO NOT RE-STERILIZE OR REPROCESS
- If any of the above prevention guidelines were not followed, or there is any reason to believe that ENDO SMARTCAP™ System was contaminated, immediately discard the entire tubing and water bottle. Replace with a new tubing and an unopened sterile water bottle.
- ALWAYS inspect the GI endoscopes o-rings and gaskets (air / water button, endoscope connection, etc) for damage. Damaged components can result in less than optimal performance of ENDO SMARTCAP™ Tubing.
- Sterility is not guaranteed if package has been opened or damaged. Do not use if packaging or product is damaged.
- ENDO SMARTCAP™ Tubing is for 24 hour use (discard daily).
- This product contains DEHP (Bis(2-ethylhexyl)phthalate) and may be harmful to sensitive populations (children and pregnant/nursing mothers.)
- Due to the short duration of use and the slow migration rate of DEHP from leachable materials into aqueous fluid the residual chemical safety risk is deemed negligible. The risk and benefits of using this device shall be carefully evaluated by a clinician on a case-by-case basis.
- PRECAUTION: If sterile water bottle needs replacing during a procedure, refrain from depressing the air/water valve. A potential for backflow may occur if the air/water valve is depressed while no sterile water bottle is connected to the tubing. If this occurs, replace the tubing immediately.
- Contact customer service for assistance.
- Federal (USA) law restricts the sale of this device by or on the order of a physician. **Rx ONLY**

Instructions for Use (by product number):

100145 / 100145U/ 100145CO2 / 100145CO2U

ENDO SMARTCAP™ Tubing for OLYMPUS® 140/240, 160/260, 180/280, 190/290 Series GI Endoscopes

STERILIZED USING
ETHYLENE OXIDE

24 HOUR USE
(DISCARD DAILY)

CONTAINS DEHP
(Bis(2-ethylhexyl)phthalate)

100150 / 100150U/ 100150CO2 / 100150CO2U

ENDO SMARTCAP™ Tubing for OLYMPUS® 100/200, 130/230 Series GI Endoscopes

STERILIZED USING
ETHYLENE OXIDE

24 HOUR USE
(DISCARD DAILY)

CONTAINS DEHP
(Bis(2-ethylhexyl)phthalate)

- Open the sterile peel pack and remove ENDO SMARTCAP™ Tubing. Open a bottle of sterile water (250cc, 500cc, or 1000cc). Drop the weighted tube into the sterile water bottle and firmly tighten the bottle cap to ensure a secure seal. Remove the label from the pouch and apply it to the bottle of sterile water. Write the date and time on the label.
- NOTE: If CO₂ source is used, connect the stopcock of ENDO SMARTCAP™ Tube to the CO₂ tubing. Follow instructions from the CO₂ manufacturer.
- Place ENDO SMARTCAP™ Tubing, connected to the bottle of sterile water, in the water bottle holder provided.
- Connect the distal tip of ENDO SMARTCAP™ Tubing to the GI endoscope with a firm pushing motion.
- Turn on the air processor or CO₂.
- Prime the channel and test for air and water prior to insertion of GI endoscope. If the water pressure is low, ensure that the water bottle is tightly closed.
- WARNING: To prevent cross-contamination, always prime the air / water channel prior to inserting the GI endoscope into the patient.
- If water bottle needs to be replaced, use proper aseptic technique.
- When the procedure is finished, close the pinch clip and disconnect ENDO SMARTCAP™ Tubing from GI endoscope. Turn off air processor/CO₂ at the conclusion of the procedure.
- Discard the ENDO SMARTCAP™ Tubing daily.

100160 / 100160U/ 100150CO2 / 100150CO2U

ENDO SMARTCAP™ Tubing for PENTAX® GI Endoscopes

STERILIZED USING
ETHYLENE OXIDE

24 HOUR USE
(DISCARD DAILY)

CONTAINS DEHP
(Bis(2-ethylhexyl)phthalate)

- Open the sterile peel pack and remove the ENDO SMARTCAP™ Tubing. Open a bottle of sterile water (250cc, 500cc, or 1000cc). Drop the weighted tube into the sterile water bottle and firmly tighten the bottle cap to ensure a secure seal. Remove the label from the pouch and apply it to the bottle of sterile water. Write the date and time on the label.
- Connect the tubing with the metal tip of the ENDO SMARTCAP™ Tubing into the GI endoscope by twisting the tip while pushing firmly.
- NOTE: If CO₂ source is used, connect the stopcock of the ENDO SMARTCAP™ Tube to the CO₂ tubing. Follow instructions from the CO₂ manufacturer.
- Connect the white air inlet tube into the processor with a firm twist until the connector clicks into place.
- Place the ENDO SMARTCAP™ Tubing, connected to the bottle of sterile water, in the water bottle holder provided.
- Turn on the air processor or CO₂.
- Prime the channel and test for air and water prior to insertion of GI endoscope. If the water pressure is low, ensure that the water bottle is tightly closed.
- WARNING: To prevent cross-contamination, always prime the air / water channel prior to inserting the GI endoscope into the patient.

9. If water bottle needs to be replaced, use proper aseptic technique.
10. When the procedure is finished, close the pinch clip and disconnect ENDO SMARTCAP™ Tubing from the GI endoscope. Turn off air processor/CO₂ at the conclusion of the procedure.
11. Discard the ENDO SMARTCAP™ Tubing daily.

100165 / 100165U/ 100165CO₂ / 100165CO₂U
ENDO SMARTCAP™ Tubing for FUJIFILM™ GI Endoscopes

STERILE EO STERILIZED USING ETHYLENE OXIDE

24 HOUR USE (DISCARD DAILY)

DEHP CONTAINS DEHP (Bis(2-ethylhexyl)phthalate)

1. Open the sterile peel pack and remove ENDO SMARTCAP™ Tubing. Open a bottle of sterile water (250cc, 500cc, or 1000cc). Drop the weighted tube into the sterile water bottle and firmly tighten the bottle cap to ensure a secure seal. Remove the label from the pouch and apply it to the bottle of sterile water. Write the date and time on the label.
2. Connect the distal tip of ENDO SMARTCAP™ Tubing to the GI endoscope. Align the groove on the connector and the pin on the scope side. Twist the connector to desired position; stop when undue force is required.
3. Place ENDO SMARTCAP™ Tubing, connected to the bottle of sterile water, in the water bottle holder provided.
4. NOTE: If CO₂ source is used, connect the stopcock of ENDO SMARTCAP™ Tube to the CO₂ tubing. Follow instructions from the CO₂ manufacturer.
5. Turn on the air processor or CO₂.
6. Prime the channel and test for air and water prior to insertion of GI endoscope. If the water pressure is low, ensure that the water bottle is tightly closed.
7. WARNING: To prevent cross-contamination, always prime the air / water channel prior to inserting the GI endoscope into the patient.
8. If water bottle needs to be replaced, use proper aseptic technique.
9. When the procedure is finished, close the pinch clip and disconnect ENDO SMARTCAP™ Tubing from GI endoscope. Turn off air processor/CO₂ at the conclusion of the procedure.
10. Discard ENDO SMARTCAP™ Tubing daily.

100141
ENDO SMARTCAP™ Adapter for FUJIFILM™ 530 Series Colonoscopes

STERILE EO STERILIZED USING ETHYLENE OXIDE

DO NOT REUSE

DO NOT RESTERILIZE

1. Open the sterile peel pack and remove the ENDO SMARTCAP™ Adapter (# 100141). Note that the 100141 ENDO SMARTCAP™ Adapter is designed for single use. Verify that the adapter's gasket is in place where the air/water bottle will connect. Inspect the colonoscope's o-rings and gaskets (air/ water button, endoscope connection, etc) for damage. Damaged components can result in less than optimal performance of the ENDO SMARTCAP™ Adapter.
2. Connect the ENDO SMARTCAP™ Adapter to the colonoscope. Align the grooves on the ENDO SMARTCAP™ Adapter and the pins on the colonoscope. The backflow valve must point down; forcing the adapter on in the other position can damage the colonoscope. Twist the adapter clockwise.
3. Attach the water bottle for the air/water source to the adapter. (Either FUJIFILM™ WT-2 water bottle or MEDIVATORS™ 100165 ENDO SMARTCAP™ Tubing.) Press the tubing connector against the adapter and turn clockwise.
4. Prime the channels and test for air, water, and forward water jet function prior to insertion of colonoscope. If the water pressure is low, ensure that all water bottles are tightly closed.
5. Firmly attach the ENDOGATOR™ Tubing's backflow valve to the adapter's backflow valve.
6. Prior to the procedure, activate the foot switch on the irrigation pump and prime the tubing and the colonoscope with sterile water.
7. WARNING: To prevent cross-contamination, always prime the air / water channel prior to inserting the GI endoscope into the patient.
8. If water bottle needs to be replaced, use proper aseptic technique.
9. WARNING: If forward water jet connector is leaking, disconnect and replace with another adapter. Contact Customer Service with the leaking adapter for replacement instructions.
10. When the procedure is finished, leave the adapter on the colonoscope until it reaches the reprocessing room.
11. Discard the adapter after each procedure. Discard the ENDO SMARTCAP™ Tubing and ENDOGATOR™ Tubing daily.

WARNING:

After use, these products may be potential biohazards. Handle and dispose of in accordance with accepted medical practice and applicable local, state, and federal laws and regulations.

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ENDO SMARTCAP™, ENDOGATOR™, and MEDIVATORS™ are trademarks of Medivators Inc.
FUJIFILM™ is a trademark of FUJIFILM Corporation.
OLYMPUS® is a registered trademark of Olympus Corporation.
PENTAX® is a registered trademark of Hoya Corporation.

Manufactured in the USA by:

Medivators
3150 Pollok Drive
Conroe, TX 77303
Tel: +1.936.539.0391

Medivators Inc.
14605 28th Avenue North
Minneapolis, MN 55447 USA
Toll Free: +1.800.444.4729

Medivators BV
Sourethweg 11
6422PC Heerlen
The Netherlands
Tel: +31.45.5.471.471

Cantel (Australia) Pty Ltd
300 Lorimer Street, Port Melbourne, Victoria 3207, Australia
P.O. Box 486 Port Melbourne, Victoria 3207, Australia
Toll Free: +61.1300.211.422
Fax: +61.3.9823.1899
Email: info@cantel.com.au