MEDIVATORS®

ENDO GATOR®

Hybrid Irrigation Tubing

REF CATALOG NUMBER

ENDO GATOR hybrid irrigation tubing (With Auxiliary Water Jet Connector) for OLYMPUS® 140/160/180 Series GI Endoscopes with OLYMPUS OFP or BOSTON SCIENTIFIC® ENDOSTAT™ II pump irrigation units #100605 (including "U" series)

ENDO GATOR hybrid irrigation tubing (With Auxiliary Water Jet Connector) for OLYMPUS 140/160/180 Series GI Endoscopes with OLYMPUS AFU-100, OLYMPUS OFP, ERBE® EIP2 or EGP-100 irrigation units #100606 (including "U" series)

Intended Use:

The ENDOGATOR hybrid irrigation tubing (tubing and accessories to accommodate various GI endoscopes and irrigation pumps) is intended to provide irrigation via sterile water and to supply air (via an air pump) or CO₂ (via a CO₂ supply) along with sterile water during GI endoscopic procedures when used in conjunction with an irrigation pump or cautery unit.

Warnings and Precautions:

- **ALWAYS** prime the air/water channel prior to inserting the GI endoscope into the patient.
 - **ALWAYS** prime the auxiliary water channel prior to inserting the GI endoscope into patient.
 - **NEVER** attach or reattach the ENDOGATOR hybrid irrigation tubing to a GI endoscope that is/was inside a patient and has not been reprocessed.
 - **ALWAYS** close the pinch clip on the air/water tube before disconnecting the air/water tubing from the GI endoscope.
 - **NEVER** turn off the air processor/CO₂ supply while the ENDOGATOR hybrid irrigation tubing is attached to a GI endoscope.
 - **DO NOT REMOVE THE BACKFLOW VALVE.** The backflow valve is permanently attached to the irrigation tubing.
 - The backflow valve attached to the ENDOGATOR hybrid irrigation tubing must be in place. If the backflow valve is missing, then discard the entire ENDOGATOR hybrid irrigation tubing, accessories and water bottle.
 - **DO NOT RESTERILIZE OR REPROCESS.**
 - If any of the above prevention guidelines were not followed, or there is any reason to believe that the hybrid irrigation tubing system was contaminated, immediately discard the entire tubing, accessories and water bottle. Replace with a new tubing and unopened sterile water bottle.
 - **ALWAYS** inspect the GI endoscope's O-rings and gaskets (air/water button, GI endoscope connection, etc.) for damage. Damaged components can result in less than optimal performance of the ENDOGATOR hybrid irrigation tubing.
 - Sterility is not guaranteed if package has been opened or damaged. Do not use if packaging or product is damaged.
 - ENDOGATOR hybrid irrigation tubing is for 24 hour use (discard daily).
 - Contact customer service for assistance.
 - This product contains DEHP (Bis(2-ethylhexyl)phthalate) and may be harmful to sensitive populations (children and pregnant/nursing mothers).
 - Due to the short duration of use and the slow migration rate of DEHP from leachable materials into aqueous fluid, the residual chemical safety risk is deemed negligible.
- The risk and benefits of using this device shall be carefully evaluated by a clinician on a case-by-case basis.
- Federal (USA) law restricts the sale of this device by or on the order of a physician. **Rx ONLY**
 - **PRECAUTION:** If sterile water bottle needs replacing during a procedure, refrain from depressing the air/water valve. There is a potential for backflow to occur if the air/ water valve is depressed while no sterile water bottle is connected to the tubing. If this occurs, replace the tubing immediately.

STERILIZED USING
ETHYLENE OXIDE

24 HOUR USE
(DISCARD DAILY)

DEHP CONTAINS DEHP
(Bis (2-ethylhexyl) phthalate)

Instructions for Use:

1. Open the sterile peel pack and remove the ENDOGATOR hybrid irrigation tubing. Open a bottle of sterile water (1000cc or 1500cc). Drop the angle cut tubing bundle into the sterile water bottle and firmly tighten the bottle cap to ensure a secure seal. Remove the 24 hour label from the pouch and apply it to the bottle of sterile water. Write the date and time on the label.

NOTE:

If CO₂ source is used, connect CO₂ input to backflow valve of CO₂ tubing input.

NOTE:

To prevent water leakage, place irrigation tubing in pump head regardless of whether irrigation is needed.

2. Place the ENDOGATOR hybrid irrigation tubing, connected to the bottle of sterile water, in the water bottle holder provided on the pump.
3. Holding both connectors in one hand, open the pump head, insert the pump tube of the irrigation tubing and close the pump head. (Note: Ensure that the pump tubing between the tubing connectors is centered, aligned, and carefully pulled taut before closing the pump head.) Take care to prevent the connectors from touching the ground.
4. Attach the auxiliary water jet connector on the ENDOGATOR hybrid irrigation tubing to the GI endoscope's auxiliary water port by engaging the threads and turning slowly until you feel resistance. (For specific instructions with regards to the connector, follow the IFU for the connector located in each shelf carton).
5. Connect the air/water connector of the ENDOGATOR hybrid irrigation tubing to the air/water port of the GI endoscope with a firm pushing motion. If using a CO₂ source, connect CO₂ tubing directly to backflow valve on ENDOGATOR hybrid irrigation tubing.
6. Prime the tubing and the auxiliary water channel prior to use by activating the foot pedal; flush water through the entire GI endoscope.
7. Turn on the air processor or CO₂.

8. Prime the channel and test for air and water prior to insertion of GI endoscope into patient. If the water pressure is low, then ensure that the water bottle is tightly closed.

WARNING:

To prevent cross-contamination, always prime the auxiliary water channel prior to inserting the GI endoscope into patient.

9. If the water bottle needs to be replaced, use proper aseptic techniques.
10. When the procedure is finished, close the pinch clip and disconnect from the GI endoscope.
11. Turn off the air processor/CO₂ source at the conclusion of the procedure. Turn the pump off before disconnecting.
12. Discard the ENDOGATOR® hybrid irrigation tubing daily. (For specific instructions with regards to the connector, follow the IFU for the connector located in each shelf carton).

WARNING:

After use, these products may be potential biohazards. Handle and dispose of in accordance with accepted medical practice and applicable local, state, and federal laws and regulations.

Manufactured in the USA by:
Medivators Inc.
3150 Pollok Drive
Conroe, TX 77303
USA
Tel: +1.936.539.0391

Medivators Inc. BV
Sourethweg 11
64422PC Heerlen
The Netherlands
Phone: +31.45.6.471.471

Medivators Inc. Headquarters
14605 28th Avenue North
Minneapolis, MN 55447 USA
Toll Free: +1.800.444.4729

C.R. KENNEDY & COMPANY
PTY LTD
300 LORIMER STREET
PORT MELBOURNE 3207
VICTORIA
AUSTRALIA
TEL : +61 3 9823 1555
FAX : +61 3 9823 1591

www.medivators.com

ENDOGATOR® and MEDIVATORS® are registered trademarks of Medivators Inc.
OLYMPUS® is a registered trademark of Olympus Corporation.
ERBE® is a registered trademark of Erbe Elektromedizin GmbH.
BOSTON SCIENTIFIC® and ENDOSTAT™ are registered trademarks and trademarks of Boston Scientific Corporation.

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www.medivators.com

Medivators Inc. Headquarters
14605 28th Avenue North
Minneapolis, MN 55447 VS
Gratis nummer: +1.800.444.4729

Medivators Inc. BV
Sourethweg 11
6422 PC Heerlen
Nederland
Telefoon: +31.45.6.471.471

C.R. KENNEDY & COMPANY PTY LTD
300 LORIMER STREET
PORT MELBOURNE 3207
VICTORIA
AUSTRALIEN
TEL.: +61 3 9823 1555
FAX: +61 3 9823 1591

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R ONLY**STERILE EO**STERILIZED USING
ETHYLENE OXIDE 24 HOUR USE
(DISCARD DAILY) **DEHP** CONTAINS DEHP
(Bis (2-ethylhexyl) phthalate)

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Medivators Inc. Headquarters
14605 28th Avenue North
Minneapolis, MN 55447 USA
Gratisnummer: +1.800.444.4729

Medivators Inc. BV
Sourethweg 11
64422PC Heerlen
Holland
Telefon: +31.45.6.471.471

C.R. KENNEDY & COMPANY PTY LTD
300 LORIMER STREET
PORT MELBOURNE 3207
VICTORIA
AUSTRALIEN
TEL.: +61 3 9823 1555
FAX: +61 3 9823 1591

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STERILE EOSTERILIZED USING
ETHYLENE OXIDE24 HOUR USE
(DISCARD DAILY)**DEHP** CONTAINS DEHP
(Bis (2-ethylhexyl) phthalate)

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Medivators Inc. Headquarters
14605 28th Avenue North
Minneapolis, MN 55447 USA
Gratisnummer: +1 800 444 4729

Medivators Inc. BV
Sourethweg 11
64422PC Heerlen
Nederland
Telefon: +31 45 6 471 471

C.R. KENNEDY & COMPANY PTY LTD
300 LORIMER STREET
PORT MELBOURNE 3207
VICTORIA
AUSTRALIEN
TEL.: +61 3 9823 1555
FAX: +61 3 9823 1591

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Sede da Medivators Inc. Inc.
ENDOGATOR® e MEDIVATORS® são marcas comerciais registradas da Medivators Inc.
OLYMPUS® é uma marca comercial registrada da Olympus Corporation.
ERBE® é uma marca registrada da Erbe Elektromedizin GmbH.
BOSTON SCIENTIFIC® e ENDOSTAT™ são marcas comerciais e marcas registradas da Boston Scientific Corporation.