Intended Use:

Mini PULL THRU channel cleaning device is a disposable device which can be used to pre-clean lumen channels.

Warnings and Precautions:


- **ALWAYS** wear gloves and other personal protective equipment while cleaning bronchoscopes or GI endoscopes as dictated by your organization's policies and procedures.
- **ALWAYS** use on wet channels.
- **DISCARD** Mini PULL THRU channel cleaning device if sharp bends in the plastic sheath are observed during insertion to avoid the risk of damaging the scope's channels.
- **NEVER** transfer a used Mini PULL THRU channel cleaning device from one bronchoscope or GI endoscope to another bronchoscope or GI endoscope, since cross-contamination could occur and result in patient complications.
- **DISPOSE** of Mini PULL THRU channel cleaning device after use in accordance with accepted medical practice and applicable local, state and federal laws and regulations.
- **WARNING:** Mini PULL THRU channel cleaning device must not be used in channels less than 1.4 mm ID. It is not to be used in channels that are greater than 2.6 mm ID.
- **NOTE:** Always use one (1) Mini PULL THRU channel cleaning device per bronchoscope or GI endoscope. This Mini PULL THRU channel cleaning device can be used multiple times per channel as required.
- **SINGLE USE:** These products are intended for **SINGLE USE ONLY**. Any institution, practitioner or third party who reprocesses and/or reuses these disposable devices is responsible for any injury or damage that results from reuse.
- Contact customer service for assistance.
- Federal (USA) law restricts the sale of this device by or on the order of a physician.

Instructions for Use (by product number):

100406 Mini PULL THRU channel cleaning device- Working Channel: 1.4-2.6 mm ID; Working Length: 220 cm approximately.

Compatible with OLYMPUS®/FUJINON®/PENTAX® Bronchoscopes and GI Endoscopes with channels in the designated range (1.4-2.6 mm ID).

1. Open the pouch and remove the Mini PULL THRU channel cleaning device from the package.
2. Insert the short tip end "1" of the PULL THRU channel cleaning device into channels in the sequence described below. Advance the PULL THRU channel cleaning device using short 2 cm strokes. Pull the device through the channels slowly to prevent spraying matter when the PULL THRU channel cleaning device head "2" exits the channel.
3. BRONCHOSCOPE SUCTION PORT: Insert the short tip end of the Mini PULL THRU channel cleaning device into the suction port. Advance the Mini PULL THRU channel cleaning device using short 2 cm strokes until it exits the distal end. Once through the distal end, grasp this end and pull the device slowly to prevent spraying matter when the Mini PULL THRU channel cleaning device head exits the channel.
4. BRONCHOSCOPE BIOPSY PORT: Insert the short tip end of the Mini PULL THRU channel cleaning device into the biopsy port. Advance the Mini PULL THRU channel cleaning device using short 2 cm strokes until it exits the distal end. Once through the distal end, grasp this end and pull the device slowly to prevent spraying matter when the Mini PULL THRU channel cleaning device head exits the channel.
5. GI ENDOSCOPE CHANNEL 1. Insert device into the suction cylinder through to the suction connector. If there is resistance, insert device through the suction connector to the suction cylinder. Once the plastic sheath "1" has exited the distal end of the suction channel, pull the device through the endoscope until it has passed completely through the suction channel.
6. GI ENDOSCOPE CHANNEL 2. Insert device into the suction cylinder through to the distal tip of endoscope. Once the plastic sheath "1" has exited the distal tip, pull the device through the endoscope until it has passed completely through this channel.
7. GI ENDOSCOPE CHANNEL 3. Insert the device into the biopsy channel port through to the distal tip of endoscope. Once the plastic sheath "1" has exited the distal tip of the biopsy channel, pull the device through the biopsy channel until it has passed completely through this channel.
8. The wipers should pass easily through the channel.
9. Dispose of device after use.

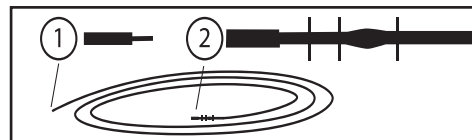WARNING:

After use, these products may be potential biohazards. Handle and dispose of in accordance with accepted medical practice and applicable local, state, and federal laws and regulations.

Manufactured for:
MEDIVATORS
3150 Pollok Drive
Conroe, TX 77303
USA
Tel: +1.936.539.0391
By: F.R. Galantai Manufacturing Limited.

www.medivators.com

MEDIVATORS Inc. Headquarters
14605 28th Avenue North
Minneapolis, MN 55447 USA
Toll Free: +1.800.444.4729

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