# MEDIVATORS®

# ENDO GATOR®

## Endoscopy Irrigation Tubing

### Reorder No. 100130 / 100130U

ENDO GATOR Tubing for BOSTON SCIENTIFIC® ENDOSTAT™ II, OLYMPUS® OFF or ENDO STRATUS® Pumps

ENDO GATOR тръбичка за помпи BOSTON SCIENTIFIC® ENDOSTAT™ II, OLYMPUS® OFF или ENDO STRATUS®

Tubo ENDO GATOR para as bombas BOSTON SCIENTIFIC® ENDOSTAT™ II, OLYMPUS® OFF ou ENDO STRATUS®

Tubulure ENDO GATOR pour pompes BOSTON SCIENTIFIC® ENDOSTAT™ II, OLYMPUS® OFF ou ENDO STRATUS®

Hadička ENDO GATOR pro pumpy BOSTON SCIENTIFIC® ENDOSTAT™ II, OLYMPUS® OFF nebo ENDO STRATUS®

### Reorder No. 100130F / 100130FU

ENDO GATOR Tubing for BOSTON SCIENTIFIC ENDOSTAT II, OLYMPUS OFF, or ENDO STRATUS Pumps with FUJINON® Connector

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### Reorder No. 200230 / 200230U

ENDO GATOR Tubing for ENDO GATOR™ EGP-100 Irrigation Pump, OLYMPUS OFF and AFU-100 and ERBE® EIP2 Pumps

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### Reorder No. 200230F / 200230FU

ENDO GATOR Tubing for ENDO GATOR EGP-100 Irrigation Pump with FUJINON Connector

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### Reorder No. 100125 / 100125F / 100110

ENDO GATOR Tubing for PENTAX® and MEDITRON Pumps

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### Reorder No. 100116

ENDO GATOR Auxiliary Water Jet Connector for PENTAX GI Endoscopes

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### Reorder No. 100241

ENDO GATOR Single Use Auxiliary Water Jet Connector for OLYMPUS GI Endoscopes

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### Reorder No. 100135

ENDO GATOR™ Irrigation Channel Tubing

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### Reorder No. 100242

ENDO GATOR Single Use Auxiliary Water Jet Connector for PENTAX GI Endoscopes

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MODE D'EMPLOI

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 **MEDIVATORS**  
A Cantel Medical Company

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Medivators BV  
Sourthweg 11  
6422PC Heerlen  
The Netherlands  
Tel: +31.45.5.471.471

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### Reorder No. 100130F / 100130FU

ENDO GATOR Tubing for BOSTON SCIENTIFIC ENDOSTAT II, OLYMPUS OFP, or ENDO STRATUS Pumps with FUJINON® Connector

Tubo ENDO GATOR per pompe BOSTON SCIENTIFIC ENDOSTAT II, OLYMPUS OFP o ENDO STRATUS con connettore FUJINON®

Tubo ENDO GATOR para bombas BOSTON SCIENTIFIC ENDOSTAT II, OLYMPUS OFP o ENDO STRATUS con conector FUJINON®

ENDO GATOR vamzdėliai skirti BOSTON SCIENTIFIC ENDOSTAT II, OLYMPUS OFP, arba ENDO STRATUS siurbliams su FUJINON® jungtimi

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### Reorder No. 200230 / 200230U

ENDO GATOR Tubing for ENDO GATOR™ EGP-100 Irrigation Pump, OLYMPUS OFP and AFU-100 and ERBE® EIP2 Pumps

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Manufactured in the USA by:

Medivators  
3150 Pollok Drive  
Conroe, TX 77303  
Tel: +1.936.539.0391

Medivators Inc. Headquarters  
14605 28th Avenue North  
Minneapolis, MN 55447 USA  
Toll Free: +1.800.444.4729

Medivators BV  
Sourethweg 11  
6422PC Heerlen  
The Netherlands  
Tel: +31.45.5.471.471

C.R. KENNEDY & COMPANY PTY  
LTD  
300 LORIMER STREET  
PORT MELBOURNE 3207  
VICTORIA  
AUSTRALIA  
TEL: +61.3.9823.1555  
FAX: +61.3.9823.1591

0050 www.medivators.com

MEDIVATORS®, ENDO STRATUS® and ENDO GATOR® are registered trademarks of Medivators Inc.  
OLYMPUS® is a registered trademark of OLYMPUS Corporation.  
FUJINON® is a registered trademark of FUJIFILM Corporation.  
PENTAX® is a registered trademark of Hoya Corporation.  
ERBE® is a registered trademark of Erbe Elektromedizin GmbH.  
BOSTON SCIENTIFIC® and ENDOSTAT™ are trademarks and registered trademarks of Boston Scientific Corporation.

Issue date: 10/15/2013

## Intended Use:

The ENDOGATOR® system (tubing and accessories to accommodate various GI endoscopes and irrigation pumps) is intended to provide irrigation via sterile water during GI endoscopic procedures when used in conjunction with an irrigation pump (or cauterly unit).

## Warnings and Precautions:

- Guidelines for Preventing Cross Contamination:
  - ALWAYS prime the auxiliary water channel prior to inserting the GI endoscope into patient.
  - NEVER attach the ENDOGATOR irrigation tubing directly to the biopsy channel without also using a secondary backflow valve. (applicable if using PENTAX® biopsy channel)
  - DO NOT REMOVE THE BACKFLOW VALVE. Backflow valves are permanently attached to the ENDOGATOR irrigation tubing.
  - The backflow valve attached to the ENDOGATOR irrigation tubing must be in place. If the backflow valve is missing, then discard the entire ENDOGATOR irrigation tubing, accessories and water bottle.
  - DO NOT RE-STERILIZE OR REPROCESS
- If any of the above prevention guidelines were not followed, or there is any reason to believe that the ENDOGATOR system was contaminated, immediately discard the entire ENDOGATOR irrigation tubing, accessories and water bottle. Replace with new ENDOGATOR irrigation tubing, accessories, and unopened sterile water bottle.
- Sterility is not guaranteed if package has been opened or damaged. Do not use if packaging or product is damaged.
- ENDOGATOR irrigation tubing is for 24 hour use (discard daily).
- This product contains DEHP (Bis(2-ethylhexyl)phthalate) and may be harmful to sensitive populations (children and pregnant/nursing mothers).
- Due to the short duration of use and the slow migration rate of DEHP from leachable materials into aqueous fluid the residual chemical safety risk is deemed negligible. The risk and benefits of using this device shall be carefully evaluated by a clinician on a case-by-case basis.**
- Contact customer service for assistance.
- Federal (USA) law restricts the sale of this device by or on the order of a physician. **RX ONLY**

## Instructions for Use (by product number):

### 100130 / 100130U

ENDOGATOR Tubing for BOSTON SCIENTIFIC® ENDOSTAT™ II, OLYMPUS® OFF, or ENDO STRATUS® Pumps

### 100130F / 100130FU

ENDOGATOR Tubing for BOSTON SCIENTIFIC ENDOSTAT II, OLYMPUS OFF, or ENDO STRATUS Pumps with FUJINON® connector

### 200230 / 200230U

ENDOGATOR Tubing for ENDOGATOR™ EGP-100 Irrigation Pump, OLYMPUS OFF and AFU-100 and ERBE® EIP2 Pumps

### 200230F / 200230FU

ENDOGATOR Tubing for ENDOGATOR EGP-100 Irrigation Pump with FUJINON Connector

- Open the sterile peel pack and remove the ENDOGATOR irrigation tubing. Open a bottle of sterile water (250cc, 500cc, or 1000cc). Drop the tube into the sterile water bottle and firmly tighten the bottle cap to ensure a secure seal. Remove the label from the pouch and apply it to the bottle of sterile water. Write the date and time on the label.
- Open pump head, insert ENDOGATOR tubing and close pump head. (Note: Ensure that tubing is placed in between tube markers.)
- Connect the backflow valve (distal tip of the ENDOGATOR tubing) to either the ENDOGATOR™ auxiliary water connector or biopsy irrigation accessory. When connecting to a FUJINON® GI endoscope, use item 200230F or item 100130F with the elbow connection.
- Prime the tubing prior to use by activating the foot pedal; flush water through the entire GI endoscope.
- WARNING: To prevent cross-contamination, always prime the auxiliary water channel prior to inserting the GI endoscope into patient.
- If the water bottle needs to be replaced, use proper aseptic technique.
- When the procedure is finished, turn off the pump unit before disconnecting the ENDOGATOR irrigation tubing.
- Discard the ENDOGATOR tubing daily.

### 100105

ENDOGATOR™ Extension Tubing

- Open the sterile peel pack and remove the ENDOGATOR extension tubing.
- Luer lock the extension tubing to the backflow valve on the ENDOGATOR irrigation tube set.
- Attach any of the ENDOGATOR irrigation accessories to the leur lock of the backflow valve on the ENDOGATOR extension tubing.
- Prior to each procedure, activate the footswitch to prime the tube set and GI endoscope with sterile water.
- When procedure is finished, turn off the pump unit before disconnecting the ENDOGATOR tubing.

### 100125 / 100125F / 100110

ENDOGATOR Tubing for PENTAX® and MEDITRON Pumps

- Open the sterile peel pack and remove the ENDOGATOR irrigation tubing. Open a bottle of sterile water (250cc, 500cc, or 1000cc). Drop the tube into the sterile water bottle and firmly tighten the bottle cap to ensure a secure seal. Remove the label from the pouch and apply it to the bottle of sterile water. Write the date and time on the label.
- Insert the "C" ring of the (#100125) tube set into the reusable ENDOGATOR™ cartridge (#100110) with the tube side out. (Note: Ensure the ring is centered and aligned properly in the cartridge or the tubing may crimp and disrupt water flow.) Gently pull the ends of the tube into the exit slots and connect the cartridge to the pump.
- Connect the backflow valve (distal tip of the ENDOGATOR tubing) to either the ENDOGATOR auxiliary water connector or biopsy irrigation accessory. Connect tubing to the GI endoscope, and prime the tubing prior to use by activating the foot pedal; flush water through the entire GI endoscope.
- If the water bottle needs to be replaced, use proper aseptic technique.
- When the procedure is finished, turn off the pump unit before disconnecting the ENDOGATOR tubing.
- Discard the ENDOGATOR tubing daily.

100115

**ENDOGATOR™ Auxiliary Water Jet Connector for OLYMPUS® GI Endoscopes**

STERILE EO STERILIZED USING ETHYLENE OXIDE 24 HOUR USE (DISCARD DAILY)

1. Open the sterile peel-pack, remove the ENDOGATOR auxiliary water jet connector, and firmly attach to the ENDOGATOR® tube set's backflow valve via the luer lock connection.
2. Attach the ENDOGATOR auxiliary water jet connector to the GI endoscope's auxiliary water port by engaging the threads and turning slowly until you feel resistance.
3. Prior to the procedure, activate the footswitch and prime the tube set and GI endoscope with sterile water.
4. WARNING: Potential Patient Safety Risk: If auxiliary water jet connector is leaking disconnect and replace with another adapter. Contact Customer Service about the leaking adapter for replacement instructions.
5. Immediately after the procedure, disconnect the ENDOGATOR auxiliary water jet connector from the GI endoscope.

100116

**ENDOGATOR Auxiliary Water Jet Connector for PENTAX® GI Endoscopes**

STERILE EO STERILIZED USING ETHYLENE OXIDE 24 HOUR USE (DISCARD DAILY)

1. Open the sterile peel-pack, remove the ENDOGATOR auxiliary water jet connector, and firmly attach to the ENDOGATOR tube set's backflow valve via the luer lock connection.
2. Attach the ENDOGATOR auxiliary water jet connector to the GI endoscope's auxiliary water port by engaging the threads and turning slowly until you feel resistance.
3. Prior to the procedure, activate the footswitch and prime the tube set and GI endoscope with sterile water.
4. WARNING: Potential Patient Safety Risk: If auxiliary water jet connector is leaking disconnect and replace with another adapter. Contact Customer Service about the leaking adapter for replacement instructions.
5. Immediately after the procedure, disconnect the ENDOGATOR auxiliary water jet connector from the GI endoscope.

100126

**ENDOGATOR™ Backflow Valve**

STERILE EO STERILIZED USING ETHYLENE OXIDE DO NOT RESTERILIZE SINGLE USE ONLY

1. Open the sterile peel-pack and remove the backflow valve.
2. Attach the backflow valve via the luer lock connector.
3. After the procedure, discard the used backflow valve.

100131

**ENDOGATOR™ Spray Catheter**

STERILE EO STERILIZED USING ETHYLENE OXIDE DO NOT RESTERILIZE SINGLE USE ONLY

1. Open the pouch and remove the ENDOGATOR Spray Catheter from the package.
2. Attach the ENDOGATOR Spray Catheter to the distal tip of the ENDOGATOR irrigation tubing at the backflow valve.
3. Prior to the procedure, activate the footswitch and prime the ENDOGATOR Spray Catheter with sterile water. Ensure that the spray pattern is obtained when consistent pressure is applied. Insert the distal tip into the biopsy channel; advance the spray catheter using short 2 cm strokes until the spray nozzle and catheter is visualized at the distal tip of the Endoscope.
4. To spray affected areas with a consistent pattern, apply steady pressure to the footswitch of the ENDOGATOR™ irrigation Pump.
5. Dispose of spray catheter at the end of the procedure.

100135

**ENDOGATOR™ Irrigation Channel Tubing**

STERILE EO STERILIZED USING ETHYLENE OXIDE DO NOT RESTERILIZE SINGLE USE ONLY

1. Open the sterile peel-pack and attach the irrigation channel tubing to the distal tip of the ENDOGATOR irrigation tubing at the backflow valve.
2. Prior to the procedure, activate the footswitch and prime the tube set and irrigation channel with sterile water. The irrigation channel tubing feeds into the GI endoscope biopsy port and delivers fluid to a specific area for irrigation.
3. After the procedure, discard the used irrigation channel tubing.

100136

**ENDOGATOR™ Channel Adapter**

STERILE EO STERILIZED USING ETHYLENE OXIDE DO NOT RESTERILIZE SINGLE USE ONLY

1. Open the sterile peel-pack and attach the channel adapter to the distal tip of the ENDOGATOR irrigation tubing at the backflow valve.
2. Insert the channel adapter into the biopsy port of the GI endoscope, and depress foot pedal for biopsy irrigation.
3. After the procedure, discard the used channel adapter, along with the attached backflow valve.

100241

**ENDOGATOR Single Use Auxiliary Water Jet Connector for OLYMPUS GI Endoscopes**

STERILE EO STERILIZED USING ETHYLENE OXIDE DO NOT RESTERILIZE SINGLE USE ONLY

1. Open the sterile peel pack and remove the ENDOGATOR single use auxiliary water Jet Connector. Attach the ENDOGATOR single use auxiliary water jet connector to the GI endoscope's auxiliary water port by engaging the threads and turning slowly until you feel resistance.
2. Firmly attach to the ENDOGATOR tube set's backflow valve via the luer lock connection.
3. Prior to the procedure, activate the footswitch and prime the tube set and GI endoscope with sterile water.
4. WARNING: Potential Patient Safety Risk: If auxiliary water jet connector is leaking disconnect and replace with another adapter. Contact Customer Service about the leaking adapter for replacement instructions.
5. Discard the ENDOGATOR single use auxiliary water jet connector after each procedure.

100242

**ENDOGATOR Single Use Auxiliary Water Jet Connector for PENTAX GI Endoscopes**

STERILE EO STERILIZED USING ETHYLENE OXIDE DO NOT RESTERILIZE SINGLE USE ONLY

1. The ENDOGATOR single use auxiliary water jet connector contains its own backflow valve, and is a disposable replacement for the PENTAX Water Jet Check Valve Adapter OE-C12 and PENTAX Irrigation Tube OF-B113.
2. If the PENTAX Water Jet Check Valve Adapter OE-C12 is installed onto the GI endoscope, remove it by unthreading it from the water port. DO NOT discard the OE-C12. This component must be used during the reprocessing of the GI endoscope.
3. Open the sterile peel pack, remove the ENDOGATOR single use auxiliary water jet connector, and attach it to the scope's auxiliary water port by inserting the nozzle end into the port and engaging the threads while turning clockwise steadily until you feel firm resistance.
4. Connect the ENDOGATOR tubing's backflow valve to the ENDOGATOR single use auxiliary water jet connector via luer lock.
5. Prior to the procedure, activate the pump footswitch and prime the ENDOGATOR tubing and GI endoscope with sterile water.
6. After the procedure, disconnect the ENDOGATOR single use auxiliary water jet connector from the GI endoscope and ENDOGATOR tubing. Dispose of the ENDOGATOR single use auxiliary water jet connector.

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