## INTENDED USE:

The ENDO SMARTCAP CO<sub>2</sub> Source Tubing with Luer Input is intended to be used with a CO<sub>2</sub> insufflator along with ENDO SMARTCAP® irrigation tubing or ENDOGATOR® hybrid irrigation tubing to supply CO<sub>2</sub> to a GI endoscope during GI endoscopic procedures.

## WARNINGS AND PRECAUTIONS:

- **Guidelines for Preventing Cross Contamination:**
  - **ALWAYS prime the air/water channel prior to inserting the GI endoscope into the patient.**
  - NEVER turn off the CO<sub>2</sub> supply while the ENDO SMARTCAP CO<sub>2</sub> Source Tubing is attached to a GI endoscope.
  - DO NOT REPROCESS.
- If any of the above prevention guidelines were not followed, or there is any reason to believe that the system was contaminated, immediately discard the entire tubing, accessories and water bottle. Replace with a new tubing and unopened sterile water bottle.
- ALWAYS turn off the CO<sub>2</sub> Insufflator between procedures.
- ALWAYS inspect the O-rings and gaskets (GI endoscope, air/water valve, suction valve, etc.) for damage. Damaged components can result in less than optimal performance.
- The ENDO SMARTCAP CO<sub>2</sub> Source Tubing is for 24 hour use (discard daily).
- Contact customer service for assistance.
- This product contains DEHP (Bis(2-ethylhexyl)phthalate) and may be harmful to sensitive populations (children and pregnant/nursing mothers).
- The product is supplied non-sterile.
- The CO<sub>2</sub> Source tubing is to be used only for GI endoscopic procedures. Do not use the tubing set for CT Colonography (Virtual Colonoscopy).
- Federal (USA) law restricts the sale of this device by or on the order of a physician.
- **PRECAUTION:** If sterile water bottle needs replacing during a procedure, refrain from depressing the air/water valve. There is a potential for back flow to occur if the air/ water valve is depressed while no sterile water bottle is connected to the tubing. If this occurs, replace the tubing immediately.

**NON STERILE**

**!** 24 HOUR USE  
(DISCARD DAILY)

**PHT** DEHP CONTAINS DEHP  
(Bis (2-ethylhexyl) phthalate)

### WARNING:

After use, these products may present a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

## INSTRUCTIONS FOR USE:

1. Open the pouch and remove the CO<sub>2</sub> source tubing.
2. Connect the male luer of CO<sub>2</sub> Source tubing to the luer connection of ENDO SMARTCAP irrigation tubing or ENDOGATOR hybrid irrigation tubing.
3. Connect the female luer of the CO<sub>2</sub> Source tubing to the CO<sub>2</sub> insufflator.
4. Turn on the CO<sub>2</sub> insufflator. Verify that the air supply from the GI endoscopic system is turned off.
5. Prime the channel and test for gas and water flow prior to insertion of GI endoscope. If the pressure is low, ensure that the water bottle is tightly closed.

### WARNING:

To prevent cross contamination, always prime the air/water channel prior to inserting the GI endoscope into the patient.

6. When procedure is completed, close the pinch clip on the ENDO SMARTCAP irrigation tubing or ENDOGATOR hybrid irrigation tubing and disconnect from the GI endoscope.
7. Turn off CO<sub>2</sub> insufflator.
8. Discard the CO<sub>2</sub> source tubing daily.

Manufactured in the USA by:

MEDIVATORS  
3150 Pollok Drive  
Conroe, TX 77303  
Tel: +1.936.539.0391

MEDIVATORS Inc. Headquarters  
14605 28th Avenue North  
Minneapolis, MN 55447 USA  
Tel Free: +1.800.444.4729

MEDIVATORS BV **EC/REP**  
Sourethweg 11  
6422PC Heerlen  
The Netherlands  
Tel: +31.45.5.471.471

C.R. KENNEDY & COMPANY  
PTY LTD  
300 LORIMER STREET  
PORT MELBOURNE 3207  
VICTORIA  
AUSTRALIA  
TEL : +61.3.9823.1555  
FAX : +61.3.9823.1591

www.medivators.com

ENDO SMARTCAP®, ENDOGATOR®, and MEDIVATORS® are registered trademarks of MEDIVATORS Inc.

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