INTENDED USE:

The single use polyp trap is designed for in-line suction retrieval of polyps during endoscopic procedures.

WARNINGS AND PRECAUTIONS:

Guidelines for Preventing Cross Contamination:

- NEVER reprocess single use polyp traps.
- ALWAYS wear gloves and other protective equipment while transferring specimen obtained in the polyp trap to pathology receptacle.
- DISCARD polyp trap if a crack or break in the basket or tubing is observed.
- DISPOSE of the polyp trap after use. Removal of tissue and mucus cannot be guaranteed and cross contamination could occur if product is re-used.
- Single use only, do not reprocess or reuse.
- Contact customer service for assistance.

INSTRUCTIONS FOR USE (BY PRODUCT NUMBER):

100101 ECONOTRAP 1 single chamber polyp trap

1. Connect suction tubing to the center port on the polyp trap lid.
2. Connect outside tubing on the lid of the polyp trap to endoscope.
3. To detach polyp trap, disconnect the suction tubing from the suction system and the polyp trap tubing from the endoscope.
4. Remove colonic tissue per customary method to fixative container.
5. When tissue samples have been transferred to a fixative container, discard the remaining portion of the polyp trap immediately.

100102 ECONOTRAP 2 double chamber polyp trap

1. Connect suction tubing to the center port on the polyp trap lid.
2. Connect outside tubing on the lid of the polyp trap to endoscope.
3. Confirm that arrow on lid is centered with the internal chamber labeled "1" as migration of lid can occur with handling and transport
4. Rotate the lid clockwise immediately following first tissue collection to ensure tissue samples are not deposited into the same chamber.
5. Repeat step #4 throughout the process until all required specimens are collected.
6. To detach polyp trap, disconnect the suction tubing from the suction system and the polyp trap tubing from the endoscope. Loosen lid on polyp trap then lift straight up so removable chambers are not inadvertently dislodged by lateral pressure of internal suction stem.
7. Remove colonic tissue per customary method to fixative container or read step #8.
8. Optional: Place polyp trap on a flat surface. With free hand, grasp chamber #1 with thumb and index finger and gently "jiggle" in a side-to-side motion freeing the chamber from the basket. Place the entire chamber with collected tissue into pre-filled fixative container.
9. To remove the remaining chamber from the basket, simply place index finger under bottom of tissue chamber with thumb on top of chamber and lift straight up and place in a fixative container.
10. When all chambers and/or tissue samples have been transferred to a fixative container, discard the remaining portion of the polyp trap immediately.

100104 THE ADVANTAGE POLYP TRAP collection container (quad)

1. Connect suction tubing to the center port on the polyp trap lid.
2. Connect outside tubing on the lid of the polyp trap to endoscope.
3. Confirm that arrow on lid is centered with the internal chamber labeled "1" as migration of lid can occur with handling and transport
4. Rotate the lid clockwise immediately following first tissue collection to ensure tissue samples are not deposited into the same chamber.
5. Repeat step #4 throughout the process until all required specimens are collected.
6. To detach polyp trap, disconnect the suction tubing from the suction system and the polyp trap tubing from the endoscope. Loosen lid on polyp trap then lift straight up so removable chambers are not inadvertently dislodged by lateral pressure of internal suction stem.
7. Remove colonic tissue per customary method to fixative container or read step #8.
8. Optional: Place polyp trap on a flat surface and place index finger over chamber's #2, #3 and #4. With free hand, grasp chamber #1 with thumb and index finger and gently "jiggle" in a side-to-side motion freeing the chamber from the basket. Place the entire chamber with collected tissue into pre-filled fixative container.
9. To remove the remaining chamber from the basket, simply place index finger under bottom of tissue chamber with thumb on top of chamber and lift straight up and place in a fixative container.
10. When all chambers and/or tissue samples have been transferred to a fixative container, discard the remaining portion of the polyp trap immediately.

A Cantel Medical Company

Manufactured in the USA by:

MEDIVATORS
3150 Pollok Drive
Conroe, TX 77303
USA
Tel: +1.936.539.0391

MEDIVATORS Inc. Headquarters
14605 28th Avenue North
Minneapolis, MN 55447 USA
Toll Free: +1.800.444.4729

MEDIVATORS BV
Sourethweg 11
64422PC Heerlen
The Netherlands
Phone: +31.45.6.471.471

www.medivators.com

THE ADVANTAGE POLYP TRAP®, ECONOTRAP™, and MEDIVATORS™ are registered trademarks and trademarks of MEDIVATORS Inc.

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