

THE 55TH ASAIO ANNUAL CONFERENCE, DALLAS, TX, May 28-30, 2009

ASSESSMENT OF HEMOCOR HEMOCONCENTRATOR IN CONTRAST MEDIA CLEARANCE

Randy M Wenthold¹, Jan Simoni², Paula Simoni³, Suzanne J London¹, Grace Simoni², John F Moeller², Ari Halldorsson² and John A Griswold.²

¹Minntech Corporation, Minneapolis, MN; ²Surgery, Texas Tech University Health Sciences Center, Lubbock, TX; ³Covenant Health System, Lubbock, TX

Diagnostic procedures with radiocontrast agents are often associated with adverse chemotoxic and idiosyncratic reactions. In fact, contrast media has become the third leading cause of hospital-acquired acute renal failure (ARF). Realizing that hemoconcentration results in better survival of patients with ARF after cardiac surgery, we conducted a series of ex vivo experiments with Hemocor HPH[®]1400 (Minntech Corp.) to evaluate its usefulness in Optiray[®]350 (Ioversol Injection 74%; Mallinckrodt Inc., St. Louis, MO) clearance from saline, human albumin, packed RBC and whole human blood. All experiments mimicked a 2 hr cardiopulmonary bypass, scaled down to a 1 L model, using standard blood and ultrafiltrate flow rates with sampling times at: 0, .25, .5, 1 and 2 hrs. Hemocor showed to be highly biocompatible with all blood components. After 30 min of dialysis, the Optiray level was reduced by 99.8% ($y=674.4 * x^{1.932} R^2=0.991$) in saline, 96.2% ($y=615.2 * x^{1.920} R^2=0.993$) in albumin, 97.8% ($y=645.4 * x^{1.926} R^2=0.992$) in RBC, and 84.7% ($y=530.5 * x^{1.898} R^2=0.993$) in whole blood. Optiray was not detectable at the 1 hr interval in all tested solutions. This data indicates that Hemocor can be an effective therapeutic strategy in preventing contrast-induced toxicity.