

Intended Use:

The ENDO SMARTCAP air input adapter with Luer Input is intended to be used with a CO₂ insufflator along with ENDO SMARTCAP® tubing or ENDOGATOR® hybrid irrigation tubing to supply air to a PENTAX® GI endoscope during GI endoscopic procedures.

Warnings and Precautions:

- **Guidelines for Preventing Cross Contamination:**
 - **ALWAYS prime the air/water channel prior to inserting the GI endoscope into the patient.**
 - NEVER turn off the CO₂ supply while the ENDO SMARTCAP CO₂ Source Tubing is attached to a GI endoscope.
 - DO NOT REPROCESS.
- If any of the above prevention guidelines were not followed, or there is any reason to believe that the system was contaminated, immediately discard the entire tubing, accessories and water bottle. Replace with a new tubing and unopened sterile water bottle.
- ALWAYS inspect the O-rings and gaskets (GI endoscope, air/water valve, suction valve, etc.) for damage. Damaged components can result in less than optimal performance.
- The ENDO SMARTCAP air input adapter is for 24 hour use (discard daily).
- Contact customer service for assistance.
- This product contains DEHP (Bis(2-ethylhexyl)phthalate) and may be harmful to sensitive populations (children and pregnant/nursing mothers).
- The product is supplied non-sterile.
- The CO₂ Source tubing is to be used only for GI endoscopic procedures. Do not use the tubing set for CT Colonography (Virtual Colonoscopy).
- Federal (USA) law restricts the sale of this device by or on the order of a physician. **Rx ONLY**
- **PRECAUTION:** If sterile water bottle needs replacing during a procedure, refrain from depressing the air/water valve. There is a potential for back flow to occur if the air/ water valve is depressed while no sterile water bottle is connected to the tubing. If this occurs, replace the tubing immediately.

NON STERILE

! 24 HOUR USE
(DISCARD DAILY)

PHT DEHP CONTAINS DEHP
(Bis (2-ethylhexyl) phthalate)

WARNING:

After use, these products may present a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

Instructions for Use:

1. Open the pouch and remove the air input adapter.
2. Connect the male luer of air input adapter to the luer connection of ENDO SMARTCAP irrigation tubing or ENDOGATOR hybrid irrigation tubing.
3. Connect the clear connector with the black o-ring of the air input adapter to the PENTAX processor.
4. Prime the channel and test for gas and water flow prior to insertion of GI endoscope. If the pressure is low, ensure that the water bottle is tightly closed.

WARNING:

To prevent cross contamination, always prime the air/water channel prior to inserting the GI endoscope into the patient.

5. When procedure is completed, close the pinch clip on the ENDO SMARTCAP irrigation tubing or ENDOGATOR hybrid irrigation tubing and disconnect from the GI endoscope.
6. Discard the air input adapter daily.



Manufactured in the USA by:

CE Medivators
 3150 Pollok Drive
 Conroe, TX 77303
 Tel: +1.939.539.0391
 www.medivators.com

Medivators Inc.
 14605 28th Avenue North
 Minneapolis, MN 55447 USA
 Toll Free: +1.800.444.4729

Medivators BV
 Sourethweg 11
 6422PC Heerlen
 The Netherlands
 Tel: +31.45.5.471.471



C.R. KENNEDY & COMPANY PTY LTD
 300 LORIMER STREET
 PORT MELBOURNE 3207
 VICTORIA, AUSTRALIA
 TEL : +61.3.9823.1555
 FAX : +61.3.9823.1591

AUS Australian
 Sponsor Address