

RENALIN[®] 100

Cold Sterilant

*The gold standard in
dialyzer reprocessing*



Reprocessing Systems. Sterilants. Disinfectants. Concentrates.

WORKING SOLUTIONS FOR OVER 35 YEARS



MEDIVATORS
A Cantel Medical Company

RENALIN® 100

Cold Sterilant

The gold standard in dialyzer reprocessing

RENALIN Cold Sterilant has been used SAFELY AND EFFECTIVELY worldwide in tens of millions of dialyzers over the past 20 years. **RENALIN 100 Cold Sterilant** – the most widely used packaging option for RENATRON® – is used with the most robust systems in dialyzer reprocessing: **the RENATRON® Dialyzer Reprocessing Apparatus** and **RENALOG® RM Data Management System**. These three products work together to give clinics a comprehensive dialyzer reprocessing solution that has become the industry gold standard.

Overview

RENALIN® 100 Cold Sterilant is a stabilized mixture of hydrogen peroxide, peracetic acid and acetic acid. RENALIN 100 Cold Sterilant is validated as a cold sterilant (not just a disinfectant) for reprocessing hollow-fiber dialyzers, and is the leading dialyzer reprocessing sterilant worldwide.

What is the difference between RENALIN and RENALIN 100?

The chemistry of RENALIN and RENALIN 100 are exactly the same. The difference is that they are used in different machines. RENALIN is used in older RENATRON and RENATRON II Systems and requires one dilution before use. RENALIN 100 is used with the RENATRON II 100 Series Systems, which do not require pre-dilution.


Medivators Renal Systems provides a complete line of process validation test strips, reprocessing accessories and training programs to support our customers in setting up the finest reprocessing programs possible.

Effectiveness

RENALIN 100 Cold Sterilant is one of the most effective biocidal products available – and has been tested against a wide range of bloodborne viruses and bacteria. Common and key pathogens in the dialysis setting such as Hepatitis B, HIV, Staphylococcus aureus and mycobacteria (surrogate for tuberculosis) are killed in the reprocessing process by RENALIN Cold Sterilant. Even against what are considered the hardest pathogens – bacterial spores – RENALIN Cold Sterilant is able to show over a six log reduction (99.9999%) kill of the organisms. For a full list of claims and testing, see the RENALIN Cold Sterilant Research Report.

Environmentally Friendly

Unlike other disinfectants such as formaldehyde, RENALIN 100 Cold Sterilant is non carcinogenic and breaks down into small molecule by-products that are not damaging for the environment. RENALIN Cold Sterilant by-products are acetic acid (which gives vinegar it's distinctive smell), oxygen and water.

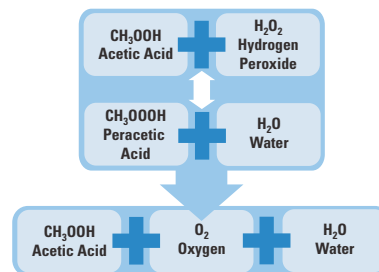

Description	Packaging Configuration	Qty of Chemical	Machine	Part Numbers
RENALIN 100 Cold Sterilant	4-1 Gallon Bottles	12 Liters	Renatron II 100 Series	78397-844 78397-845 (with 1 Test Strip Kit) 78397-846 (with 3 Test Strip Kits)
RENALIN Cold Sterilant	2-2.5 Gallon Bottles	4 Liters	Renatron, Renatron II	78335-000 78403-000 (with 3 Test Strip Kits)
RENALIN Quarts	12-1 Quart Bottles	11.355 Liters	Renatron, Renatron II	78336-000 78404-000 (with 1 Test Strip Kit)

About Medivators – Medivators Renal Systems has been providing WORKING SOLUTIONS to the hemodialysis community since 1972. Medivators Renal Systems reprocessing systems, sterilants, disinfectants and dialysate concentrates have been used in thousands of clinics and tens of millions of treatments. Today, Medivators Renal Systems continues to provide dialysis clinics with the highest quality products and services to ensure that clinics have the WORKING SOLUTIONS necessary for optimal patient care.

For more information contact your Medivators Representative: (800) 328-3340

RENALIN®, RENATRON® and RENALOG® are registered trademarks of Medivators Inc.



Medivators Inc.
14605 28th Avenue North
Minneapolis, MN
55447-4822 USA
Tel: +1.763.553.3300
Fax: +1.763.553.3387

Medivators BV
Sourethweg 11
6422 PC Heerlen
The Netherlands
Tel: +31.45.5.471.471
Fax: +31.45.5.429.595

Cantel Medical (UK) Limited
Wolsley House
Staffordshire Technology Park
Beaconside, Stafford ST18 0GA
United Kingdom
Tel: +44 (0) 1785.782420
Fax: +44 (0) 1785.782421

Cantel Medical Asia/Pacific Pte. Ltd.
1A International Business Park
#05-01 Singapore 609933
Tel: +65.6227.9698
Fax: +65.6225.6848

Cantel Medical Devices (China) Co., Ltd.
Unit 804-805, Innov Tower Block A,
1801 Hongmei Road, Xuhui 200233,
Shanghai, China
Tel: +8621.6016.1380
Fax: +8621.6121.0913